


Baby Ivy

Directions for Use

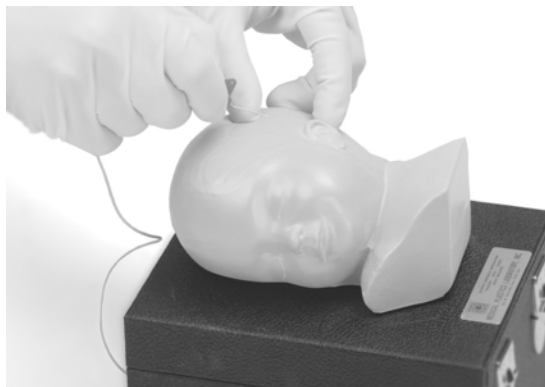

Laerdal[™]
helping save lives

Table of Contents

Instructions for Use	4
Care and Maintenance	5

Baby Ivy is a lifelike model of a newborn head. It is specifically designed for training professionals in the practice of scalp vein catheter placement.

As a result of quality construction and the ease of replacing individual parts, this simulator should provide many years of training when reasonable care and maintenance is practiced.

Item included:

- (1) Baby Ivy
- (1) Laerdal Global Warranty Booklet
- (1) Directions for Use


Instructions for Use:

1. Open lid and fill blood reservoir with water.
2. Add blood concentrate until desired blood color is achieved.
3. Close valve on bleeder tube.
4. Close valve on pump bulb.
5. To pressurize system, pump bulb two or three times to slightly inflate balloon.
6. Release valve on bleeder tube slowly and drain fluid into a cup until all lines are free of air.
7. Close lid and begin procedure.

To drain system:

1. Open valve on pump bulb.
2. Open valve on bleeder tube and close the lid.

The fluid will drain back into the blood reservoir.

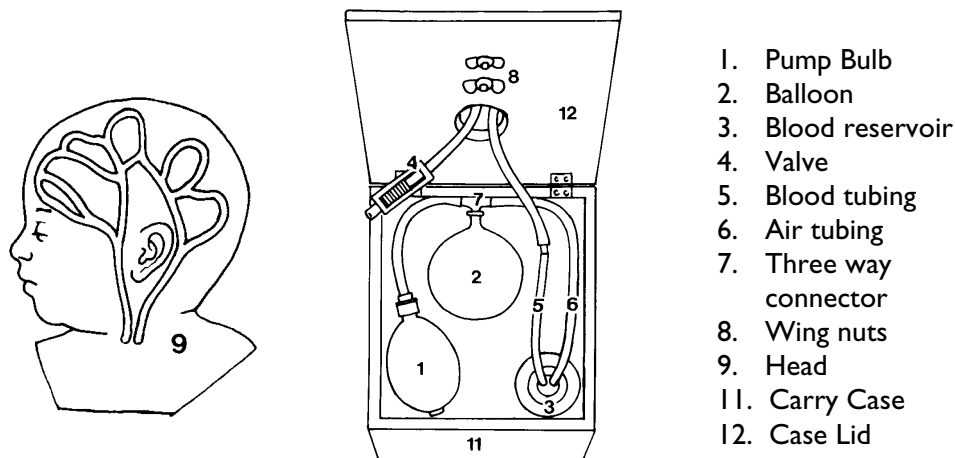

NOTE: Before long-term storage, insert a syringe into line at end of bleeder tube and inject air to force any remaining fluid out of line and into blood reservoir.

Care and Maintenance:

1. Clean with mild soap and water; do not submerge in cleaning fluids or water.
2. Use only on clean surface. Avoid felt tipped markers, ink pens, acetone, iodine or other staining products and avoid placing the trainer on newsprint or inked lines of any kind.
3. To ensure longevity, trainer should be cleaned after each training session and a general inspection should be conducted regularly.
4. Modules and all other parts should be drained and air-dried thoroughly before storage and disinfected when needed.
5. Articulating parts will benefit from a light application of talcum powder prior to training sessions.
6. Store properly between teaching sessions.

Replacement Parts:

Part Number:

375-30150

375-30250

300-00750

Description:

Head Replacement

Replacement Case & Tubing Assembly

Simulated Blood Concentrate

© Laerdal Medical, Gatesville TX. All rights reserved
1001655 rev B
October 25, 2006