

Automated External Defibrillator

Defi 5C

Automatic External Defibrillator (AED) is a lightweight, safe and battery powered device. It is design for simple and reliable operation of trained users. It can be used in situations where there could be several minutes before the arrival of advanced life support (ALS) personnel.

Easy to use

Only two buttons to operate the unit, a green ON/OFF to turn on the AED and the yellow one to deliver the shock to the victim. With the clear and simple audio and visual prompts, the Defi 5c will guide you confidently through the defibrillation step by step

Wide-range environments

In order to save the SCA victims in the wide-range environments, the Defi 5c is design to meet the requirements of IP55.

Clear Voice prompt

The clear, real human voice will reduce the stress when people is saving a SCA victim. No hesitation, just follow the voice prompt, to do the each step of CPR and defibrillation firmly and confidently.

Light Weight

Only 1.9Kg weight, the lightweight and compact size design help people to carry the Defi 5c to the victim's side fast.

Easy to Record and Transfer Data

Data from the unit can be recorded or retrieved from the device using a infra-Red

Selftest

test itself each time and weekly/monthly.



Meditech Equipment Co.,Ltd (Meditech Group)

Address: , 86 Laoshan Road,B 69 .Qingdao, Shandong Province, P. R. China
Office: 1503-C02A,30 HK Road, Qingdao, Shandong Province, P. R. China

Tel: (+86) 532 85832673 81705331
Fax: (+86) 532 81705332
Email: sales@meditech.cn

www.meditech.com.cn



MEDITECH GROUP
MEDITECH[®]

Defi 5c

A reliable algorithm is essential for deliver a shock quickly and properly!

The **Meditech** analysis algorithm was developed and tested to ensure that its sensitivity (ability to detect shock able rhythms) and the specificity (ability to detect non-shock able rhythms) exceeded the AHA and AAMI DF80 recommendations.



Technical Specifications

Type:	Automatic External Defibrillator
Self-test:	Daily, Weekly, Monthly
Mode:	Adult, Child
Waveform:	Biphasic truncated exponential
Energy:	200-Joules max.
Energy Sequence:	Adult mode: 150J, 150J, 200J 120J, 120J, 150J Children mode: 50J, 50J, 75J 30J, 30J, 50J
Charge time: (New, at 25°C)	Less than 6 sec. to 150J Less than 8 sec. to 200J
Voice Prompt:	Extensive voice prompt
Visual Indicators:	LED prompts
Control:	Two buttons: ON/OFF, Shock
ECG storage:	1,500 events.
Data transmission	Infrared

Battery

Power:	12V, 3000mAh
Type:	Non-rechargeable Li-MnO ₂ Cell

Environmental

Operating Temperature	0°C to 40°C
Operating Humidity	Relative humidity between 30% and 95% (non-condensing)
Storage Temperature	-20°C to 55°C
Storage Humidity	Up to 93% (non-condensing)

Physical

Dimensions	Depth:256mm, Width: 220mm Height:(L:65mm, H:80mm)
Weight (with battery):	1.9Kg (4.2 Pounds)



- **First** level energy: 150J
- **Second** level energy: 150J
- **Third** level energy: 200J

- Check Pads Defibrillator pads are not properly attached to the patient or properly connected to the Defi 5c.
- Caution Do not touch or move the patient
- Analyzing ECG leads are properly connected and the system is analyzing the patient's ECG.



- The Defi 5c prompt feature provides instructional prompts to guide the user through the defibrillation process. The Defi 5c Automated External Defibrillator provides audio instructions through the built-in speaker to provide operating instruction and assist the user during defibrillation



- Power LED is bright Indicates the charge left in the battery is full.
- Power LED flashes slowly Indicates the charge left in the battery is partially depleted.
- Power LED flashes fast Indicates the charge left in the battery is low



- Device has an internal memory, and the ECG signals and events can be transferred to the computer by Infra Red

• Defi 5 T

AED trainer with same outlook



MDC-W1

Material: Metal+ Plastic glass	Sound alarm
Size: 38x42x15.5mm	Carton size: 440x400x240mm
Color: any color available	Gross weight: 7.0kg




MDC-W2

Material: Metal+ Plastic glass	Adjustable Heating System
Size: 480mmx380mmx240mm	Carton size: 510x410x260mm
Color: any color available	Gross weight: 7.5kg
Sound alarm	



MDC-W3

Material: Metal+ Plastic glass	Sound alarm
Size: 480 x 38 x 240mm	Carton Size :510 x 410 x 260mm
Color: any color available	Gross weight: 6.4kg

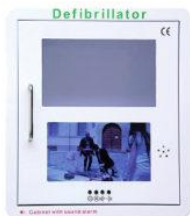

MDC-W4

Material: Metal+ Plastic glass	Video Screen (200mmx110mm)
Size: 36x40x20mm	Carton size: 435x395x235mm
Color: any color available	Gross weight: 8.5kg
Sound & Light Alarm	

We offer alarmed, custom, and generic AED wall cabinets. We also offer recessed, semi-recessed, and surface-mounted cabinets for indoor use and outdoor

If you need a total solution for your AED needs, including a AED wall cabinet, AED sign, AED Accessories and more, then contact our sales team.